Disinfectant and drying agent bags

For Care of Comprehenso AmoVes 200 Phantom Systems

To be opened **after use** of the phantom system

Keep out of reach of children | Follow the instructions for use

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Content

This Clean & Dry Set was developed for use with **Comprehenso AmoVes 200 phantom systems**. It contains **one disinfectant tablet** for the preparation of a disinfection solution as well as **two drying agent bags** for transporting and storing the phantom system.

Disinfectant Tablet

One Tablet (1g) contains 500mg Troclosesene Sodium / Sodium dichloroisocyanurate (CAS 2893-78-9)

Instructions: Use a glass or plastic container to prepare the disinfectant solution for phantom system disinfection. Dissolve one sterilizing tablet in 2.5 liters cold or hand warm tap water. Perform the disinfection procedure according to the phantom system manual. Contact time of the disinfectant solution: 15 minutes. Bactericidal according to EN 1276; Fungicide according to EN 1650. The disinfectant solution can be disposed of via wastewater after use.

Warning: May cause serious eye irritation or damage; may cause severe skin burns, may intensify fire (oxidiser); may cause respiratory irritation; Very toxic to aquatic life with long lasting effects; Do not use together with other products: May release dangerous gases (Chlorine); Contact with acids liberates toxic gas.

Precaution: Store in a cool, dry place. Keep out of reach of children. Not to be taken. Avoid contact with eyes. Use only in a well-ventilated area. Avoid release to the environment. Do not use a metallic container to prepare the solution and do not place metallic objects in the solution. If medical advice is needed, have this information at hand. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing, if eye irritation persists, get medical advice. Take care not to splash fabrics as the solution may bleach fabrics. Dispose of contents and container in accordance with applicable regulations.

Produced by: Laboratoires Polivé, 105 rue Anatole France, 92300 Levallois, France

Drying Agent bags

One Drying Agent bag contains 4g Silica Gel

Instructions: To take up any remaining liquid residue in the phantom, one or two unused drying agent bags are placed in the case during transportation and storage. Dispose of with residual waste after use.

Precaution: Do not eat. Keep away from children.

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Produced by: Tropack Packmittel GmbH, Vor dem Polstück 8, 35633 Lahnau
Hersteller: Tropic Packmittel GmbH, Vor dem Polstück 8, 35633 Lahnau
Comprehenso GmbH